

# ULCERKIT®

## THE REVOLUTION IN THE TREATMENT OF VENOUS ULCERS



Ulcerkit® is a therapeutic compression stocking system indicated for leg ulcers, easy to use improves patients' compliance. Gloria Ulcerkit® can also be used as a normal medical stocking when a certain effort for application is required.

### COMPRESSION CLASSES:

UlcerKit® is available in **Class 1** and **2**. As the system is composed by two separated parts, in case of need of a higher compression near the ankle (where usually the ulcer is located), there is the possibility to combine the lower part in CL. 2 with the upper part in CL. 1.

So the range is the following:

- **UlcerKit® 20/20:** Ankle CL. 1, Calf CL. 1
- **UlcerKit® 30/20:** Ankle CL. 2, Calf CL. 1
- **UlcerKit® 30/30:** Ankle CL. 2, Calf CL. 2

### INDICATIONS:

- Class 1:** Treatment of venous ulcer associated with other compression aids. Chronic venous insufficiency, post phlebotic, post ulcer, after sclerotherapy. Prevention of recidivism. Initial varices during pregnancy.
- Class 2/1:** Treatment of venous ulcer when a higher ankle compression is required. Stronger action at the ankle and moderate at the calf.
- Class 2:** Same as class 1 and 1/2, but with a highest compression on the leg, edema.

### CONTRAINDICATIONS:

Do not wear UlcerKit® in presence of severe arterial venous insufficiency (frequent in diabetics). In less severe cases UlcerKit® can be used and if not suitable for the patient the stocking can be removed.

### Patented Product

IT 1319230 – EU 1337211

**CIRCUMFERENCE MEASUREMENTS (CM)**

|    | S      |       | M      |       | L      |       | XL     |       |
|----|--------|-------|--------|-------|--------|-------|--------|-------|
|    | normal | max   | normal | max   | normal | max   | normal | max   |
| cC | 30-33  | 34-37 | 34-37  | 38-41 | 38-41  | 42-45 | 42-45  | 46-49 |
| cB | 19-21  |       | 22-24  |       | 25-27  |       | 28-30  |       |

**LENGTH MEASUREMENTS (CM)**

**ONE LENGTH**